


# PREGNOSTX

## ON-SITE PREGNANCY TESTS


### Instructions

To ensure accurate results, please carefully read the following instructions before starting the test.

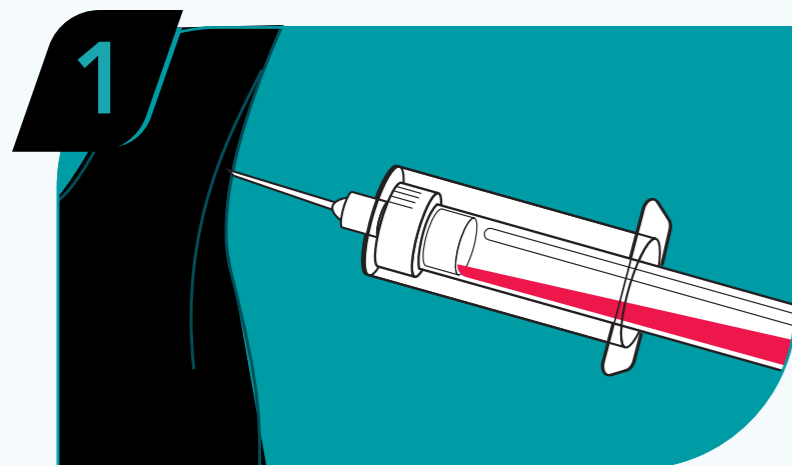


#### 1 DRAW BLOOD SAMPLE

Use the syringe to draw blood from under the tail of the test subject.

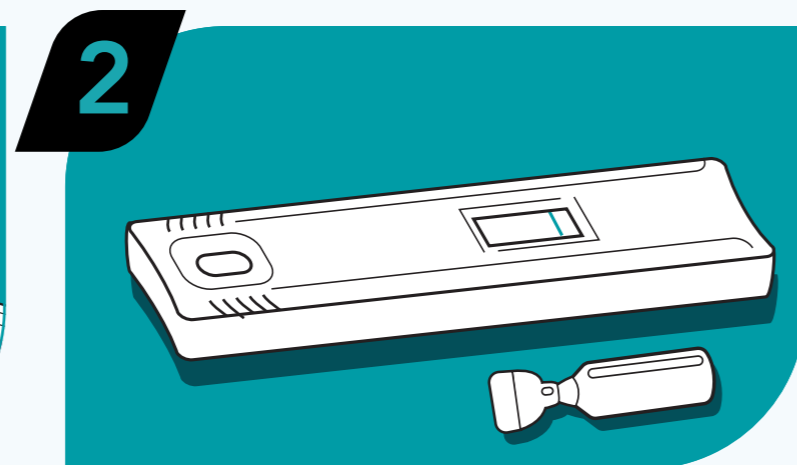


#### 2 PREPARE TEST

Lay the test flat on a clean surface.

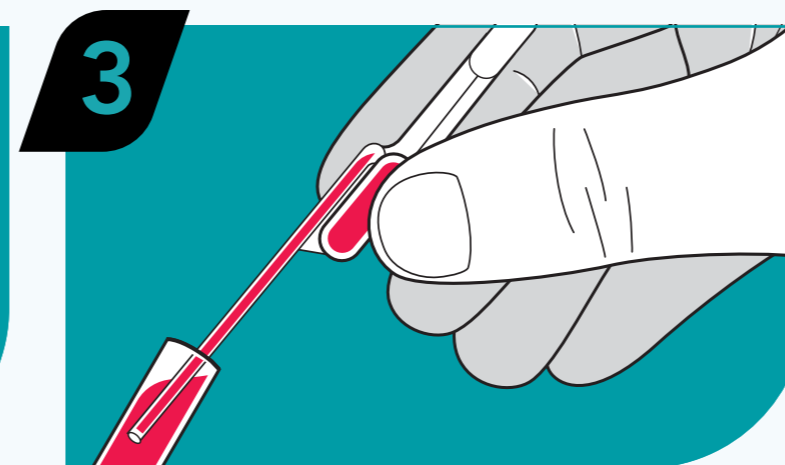


#### 3 TRANSFER SAMPLE

Use the pipette to collect the blood sample from the vacuum tube.

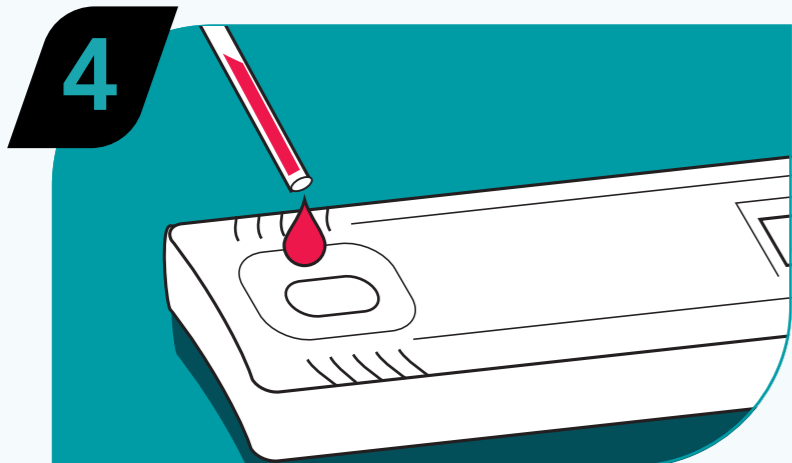


#### 4 PLACE SAMPLE

Squeeze the blood sample from the pipette into the sample port of the test.

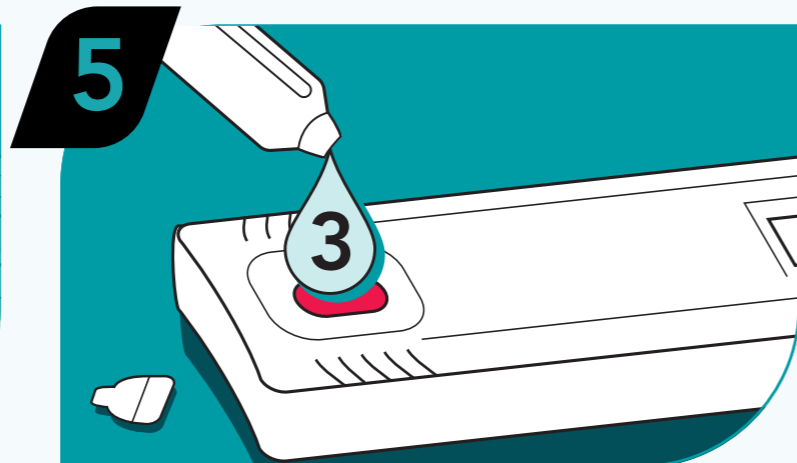


#### 5 ADD CHASE

Uncap the chase capsule and squeeze 3 drops of the liquid into the sample port.

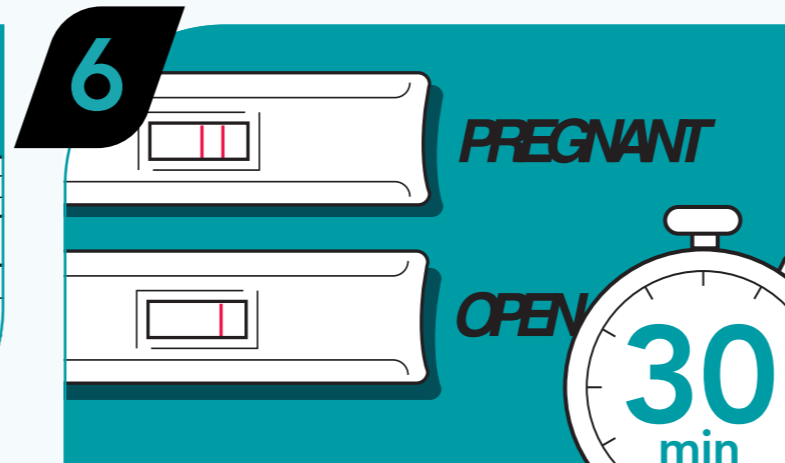

#### 6 RESULTS AFTER 30 MIN

If two lines appear, the subject is pregnant. If only one line appears, the subject is open.

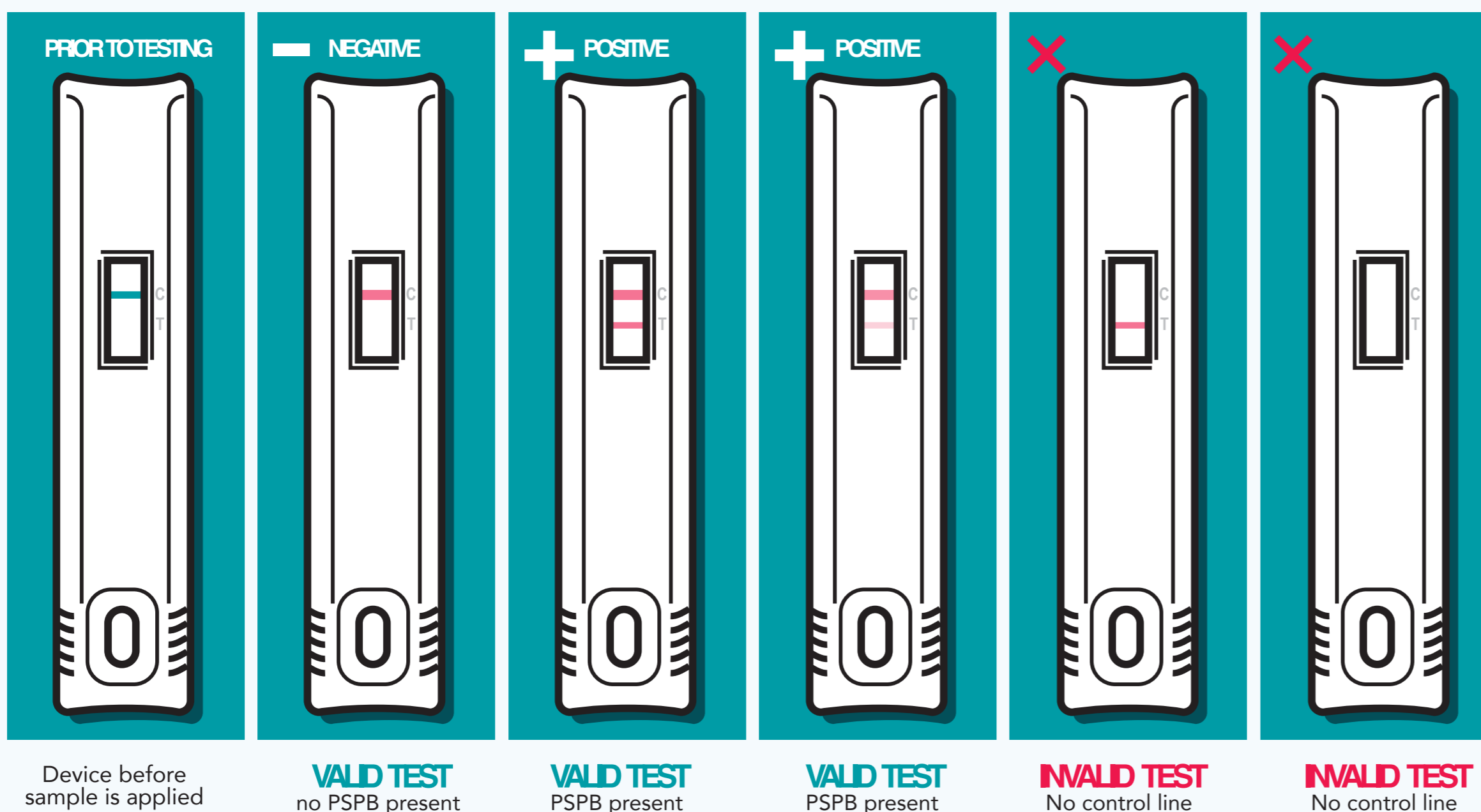

#### HOW TO READ THE RESULTS

- ✓ The cow is pregnant if the test line (T) appears after the sample and chase are added
- ✓ The cow is pregnant whether the T line appears fully, partially, bold or slightly faded
- ✓ The cow is open (not pregnant) if the T line does not appear

Find Out If She's Pregnant —Fast!